SPORT-ELEC®

Body Beautiful

training for
beauty

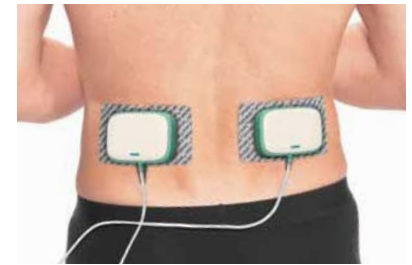
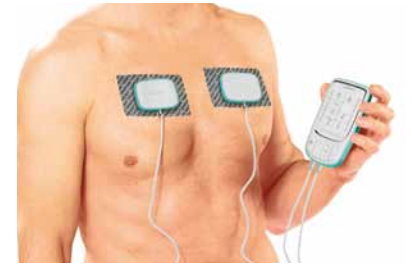
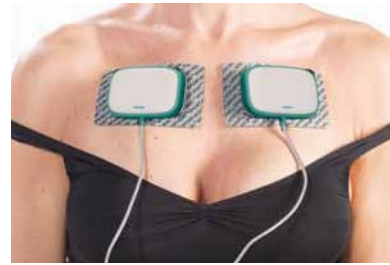
8 programmes
homme - femme

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


DESCRIPTION DE L'APPAREIL

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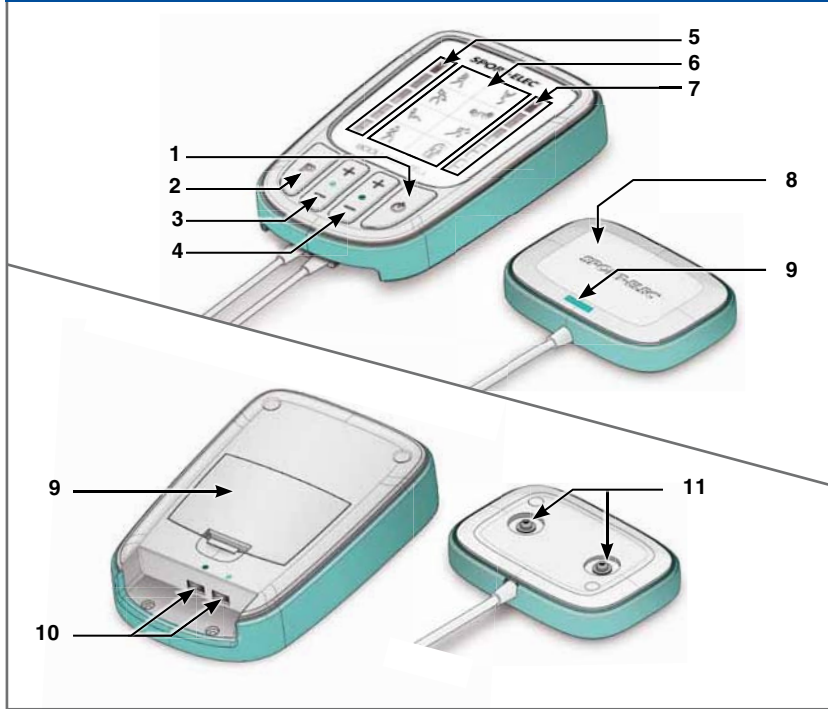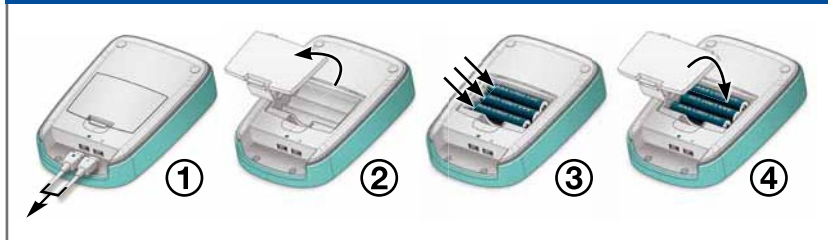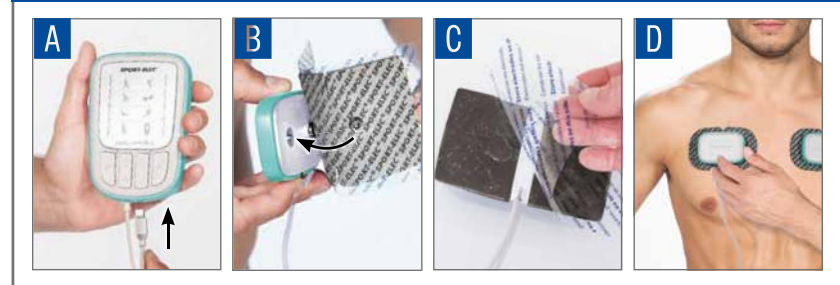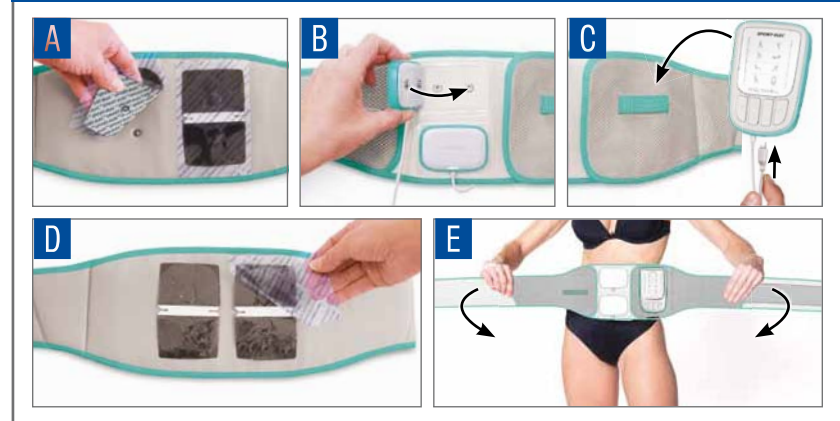
INFORMATION IMPORTANTE RELATIVE AUX BATTERIES

x3 AAA LR03 (1.5 V)

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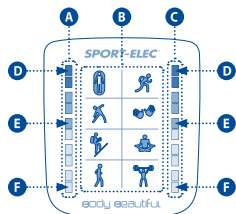
BODY BEAUTIFUL

DESCRIPTION OF THE UNIT (SEE CAPTIONED DIAGRAMS ON PAGE 3)

- ON/OFF key / Emergency stop.
- Key Programs.
- Button to adjust the power of A channel
+ : to increase the power (from 0 to 72).
- : to decrease the power (from 72 to 0).
- Button to adjust the power of B channel
+ : to increase the power (from 0 to 72).
- : to decrease the power (from 72 to 0).
- Indicator of A channel level power.
- Indicator of program.
- Indicator of B channel level power.
- Connexion module
- Colour module (or channel) indicator.
- Compartment for the batteries.
- Connection sockets to plug the 2 modules.
- Fastener buttons.

DESCRIPTION OF THE SCREEN

- A** Indicator of A channel level power.
- B** Indicator of program.
- C** Indicator of B channel level power.
- D** "Used" batteries: two red LED blink. Your device stops, even if a program is running. Change the batteries.
- E** "Weak" batteries indicator: two orange LED blink. Your device is still on operation.
- F** Disconnection warning light : blinks when the electrodes are disconnected

BEFORE USING THIS UNIT

You have just acquired an effective and powerful professional unit. Take your time to discover how to use it.

- Read the instructions carefully
- Before use, insert batteries in your device.
- Test your unit on your biceps using the quick test procedure below

THERAPEUTIC APPLICATIONS

- Private:** to strengthen and / or to maintain muscle strength.
- Physiotherapists:** to ease muscular tensions.
- In hospital environment:** to prevent muscular atrophy and to treat pain.

OPERATION QUICK TEST

Testing your unit on your biceps :

- Connect the 2 electrodes to your device (see "Connection of the device to the adhesive electrodes").
- Test the device with the program PG02.
- Increase power on this channel gradually.
- You would begin to feel contractions, the device is working.
- This discovery procedure shows your capacity in using the unit's level of performance

PROGRAMS

PROG 1 PREPARATION

20 mins prior to sport to prepare and warm up your muscles. The frequencies vary from 1 to 13 Hz to provide a genuine massage effect.
Duration : 20 min

PROG 2 DEVELOPMENT

This program allows to improve the assets on the level of muscular form.
Duration : 40 min

PROG 3 FITNESS

Use this program to maintain the level of muscle endurance, resistance, and volume acquired.
Duration : 20 min

PROG 4 SUDOLORIS

This program relieves the muscular tensions effectively.
Duration : 20 min

PROG 5 SILHOUETTE SCULPT

40 mins of body building to firm up, but also to increase muscular volume after a period of inactivity. This programme is ideal for keeping the body firm and for re-shaping your silhouette. Frequencies vary from 30 to 60 Hz alternated with frequencies at 10 Hz and 3 Hz for active recovery to build up muscle resistance.
Duration : 40 min

PROG 6 RECOVERY

Thanks to fast frequencies changes this program allows the muscles to recover faster through relaxing effects.
Duration : 20 min

PROG 7 RELAXING MASSAGE

20 mins to relieve muscular fatigue and allow you to recover after physical effort. The frequencies vary from 40 to 100 Hz to provide the state of wellbeing of a relaxing massage.
Duration : 20 min

PROG 8 MUSCLE TONIC

20 mins of more strenuous weight lifting to work on target areas. Frequencies vary from 70 to 100 Hz alternated with frequencies at 3 Hz for active recovery to build up tonicity and muscular strength.
Duration : 20 min

I. INSERTION OF BATTERIES (SEE 1, 2, 3, 4 DIAGRAMS ON PAGE 3)

- Check that the device is switched off.
- Open the batteries compartment, insert three 1,5 V AAA LR03 batteries into the compartment then to close the case.
- Comply with + and - indications.
- Do not mix used and good batteries.
- Do not use rechargeable batteries.
- For an optimal use of your device, it is advised to use good quality alkaline batteries.

II. STARTING THE UNIT

A / PRESENTATION OF THE UNIT

Your electromusculature device is an high-tech appliance, it meets the current requirements in the field of medical application. The interface "man machine" was simplified to the maximum; it is realized from lightened pictograms, indicators and function key. Among the eight pictograms the one enlightened in green displays you the program chosen by the "P" key.

You will find also 4 control keys (see diagram page 3) :

- 1 ON/OFF key (key red ) - press shortly
 - 1 "P" key to choose the programs - press shortly
 - 2 double keys "+" / "-" to adjust the power of stimulation (press shortly).
- A long press (1 second) on the key "-" returns the chosen power rapidly to 0.

B / USING THE UNIT

STEP 1 - ON / OFF "⏻"

- A short press on the "⏻" key puts the device under or except power.
- Every time when the power is on, the device carries out an auto-test on 11 essential safety parameters. During the auto-test all the indicators are lit during ½ second.
- If an internal defect in the device is detected, all the pictograms flash in green and the device stops. Then, return the device to After Sales Service.
- The auto-test does not take into account the detection of the used batteries. When batteries are used, the auto-test is not activated.
- A long press (of more than 3 seconds) involves the release of the auto-test. In this case stop the device, please, by pressing the key "⏻" and then make a short press on the key "⏻".

STEP 2 - PROGRAM SELECTION KEY "⏮"

To select a program, make one or more short presses on the key "⏮", the programs scroll. The programs are announced by pictograms.

As soon as a program is selected, corresponding pictogram lights.

Once the program is chosen, you can start it by increasing the power on one of power keys "+".

During the program the key "⏮" is inactive.

STEP 3 - END OF THE PROGRAM - STOP DURING THE PROGRAM

At the end of the session the device stops automatically.

To stop the device during the program, put the 2 power keys back on zero or press the key "⏻".

STEP 4 - ADJUSTMENT OF THE POWER

The power is divided into 10 levels: 5 green, 3 orange and 2 red levels.

To change the level press the key "+".

You have 72 levels to regulate the power from 0 to 72.

STEP 5 - BATTERIES

- If the batteries are "good", there is no blinking.
- If the batteries are "weak", two orange indicators blink. Your device still functions.
- If the batteries are "used", two red indicators blink. Your device stops, even in the course of a program. Change the batteries.

STEP 6 - DISCONNECTION

Your device is equipped with a detector of electrodes disconnection.

If the contact between the human body and a single electrode is absent, the power is stopped on the concerned channel (channel A = orange / channel B = green).

You are warned of a disconnection by blinking of green indicator during 30 seconds, after 30 seconds the blinking stops.

If none of the electrodes is in contact with the body, the power cannot be emitted.

Start your device by following the «Starting the device» procedure.

Contractions should be firm but must never cause any pain.

If the electrodes cause you any discomfort: reduce the power levels, move the electrode(s) and then gradually increase the power. Also ensure that the electrodes are sufficiently moistened.

Now that the settings have been optimized, let your device get to work for the full duration of the program.

END OF PROGRAM:

Once the program has finished, switch off the device by pressing the "⏻" button, then disconnect the modules.

Gently remove the electrodes from your skin.

Moisten them slightly with a few drops of water and replace them carefully onto their transparent plastic backing sheet.

Keep the adhesive electrodes in a plastic bag in a cool place (for example, a separate compartment in your refrigerator).

B. USING THE DEVICE WITH THE BELT

The belt has 2 pairs of snap fasteners onto which 2 wireless electrodes specially designed by SPORT-ELEC® are clipped. To obtain new electrodes, contact your usual retailer or order them from our website www.sport-elec.com.

WARNING!

Never use the belt without attaching the 2 electrodes provided.

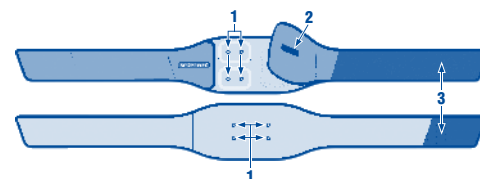
Only use electrodes supplied by SPORT-ELEC®.

PRESENTATION OF THE BELT:

1 = Snap fasteners

2 = Wire storage flap
(with a lopp fixing device)

3 = Partie auto-agrippante

CONNECTION AND POSITIONING OF THE BELT.

- ▶ Check that your device is switched off.
- ▶ Open out the belt.
- ▶ Fix the electrodes to the inside of the belt using the snap fasteners.
- ▶ Place the belt around the waist and adjust it using the Velcro strip, so that there is a good contact between the electrodes and the skin. If required, use the belt extension.
- ▶ Once the belt is in position, fix 2 modules onto the snap fasteners.
- ▶ If you experience any tingling, switch off the device and check that the electrode is in contact with the skin. If the problem persists, contact the after-sales service: sav@sport-elec.com.

STARTING THE PROGRAM WITH THE BELT

Once your device is connected, make yourself comfortable and switch it on following the «Starting the device» procedure.

You can carry out your normal activities while using the device.

END OF PROGRAM

Once the program has finished, switch off the device and unplug it from the belt.

Remove the electrodes and moisten them slightly with a few drops of water before replacing them onto their transparent plastic backing sheets.

IV. PRECAUTIONS

- ▶ Hypersensitivity to electro-stimulation may cause a skin reaction in certain individuals.
- ▶ Keep out of the reach of children or mentally-impaired persons. This product contains small parts which can be easily swallowed.
- ▶ Only use BodyBeautiful with the accessories supplied and recommended by SPORT-ELEC® INSTITUT.
- ▶ Do not use BodyBeautiful when driving, operating machinery, or during any activity where electro-stimulation could result in the risk of injury to the user.
- ▶ Always position the belt in accordance with the information provided in this manual.

III. GENERAL INSTRUCTIONS

A. USING THE DEVICE WITH THE ELECTRODES

This function allows you to use the adhesive electrodes by connecting them directly to the modules.

You can use the device with 1 or 2 modules.

GETTING STARTED

- ▶ Connect each module to the unit by matching the color indicators (photo A page 4)
- ▶ Fix the electrodes to the module(s) using the snap fasteners (photo B page 4)

POSITIONING OF THE ELECTRODES

- ▶ Remove the plastic backing sheet from the electrodes (photos C page 4).
Moisten them by spreading a few drops of water onto the adhesive gel using your fingertips and wait 2 minutes.
- ▶ Position the electrode(s) onto your body.
NB: The electrodes are for personal use. For hygiene reasons, they must always be used by the same person.

STARTING THE PROGRAM:

- ▶ Do not apply the electrodes to the lower part of your neck. Muscle contraction may be very strong and cause respiratory problems, as well as affect heart rate and blood pressure
- ▶ Do not apply to the heart area, as electro-stimulation may cause heart rhythm disturbances.
- ▶ Do not place electrodes on the face, head or temples. The effects of electro-stimulation on the brain are unknown.
- ▶ Only apply the electrodes to clean and healthy skin. Do not carry out electro-stimulation on an open wound or on swollen, infected or inflamed areas, or on skin rashes (e.g. phlebitis, thrombophlebitis, varicose veins, etc.)
- ▶ Do not use in the bath, shower or for any activity in contact with water.
- ▶ Do not use when sleeping.
- ▶ Do not use if you are fitted with an IUD contraceptive device, as contraceptive efficiency under electro-stimulation has not been established.
- ▶ Do not use on parts of the body other than those for which the accessories have been designed.
- ▶ The electrodes are for personal use. For hygiene reasons, they must always be used by the same person.

Special precautions should be taken in the following circumstances:

- ▶ Persons with suspected epilepsy
- ▶ If you have a tendency to internal bleeding or bleeding following an injury or fracture.
- ▶ Following a surgical operation. (Electro-stimulation may disrupt the recovery procedure.)
- ▶ In areas of the skin where sensitivity is impaired.

ADVERSE EFFECTS

- ▶ Skin irritation.
- ▶ Mild burns to the skin where the electrodes are applied.

MEDICAL ADVICE

Seek the advice of your doctor in the following cases:

- ▶ You suffer from serious back pain.
- ▶ You have a serious illness not mentioned in this manual.
- ▶ You have recently undergone a surgical procedure.
- ▶ You take insulin for diabetes.

RECOMMENDATIONS FOR USE

- ▶ You can restart a program cycle on another muscle zone.
- ▶ For optimum use, refer to the diagrams shown in this manual.
- ▶ The units can be cleaned using a cloth dampened with a disinfectant solution.

REPAIRS

If your device is not working correctly, stop using it and contact your retailer.

RECYCLING

Please respect national regulations when you dispose of your product when it reaches the end of its life.

Please take it to a suitable place in order to ensure that it is disposed of in a safe and environmentally-friendly manner.

CONTACT SPORT-ELEC® INSTITUT OR AN AUTHORIZED DISTRIBUTOR IF:

- ▶ You experience any irritation, skin reaction, hypersensitivity or any other reaction. However, please note that any redness of the skin on contact with the electrodes is normal and disappears shortly after using your belt.
- ▶ Correct use of the electro-stimulation belt should not cause any discomfort. SPORT-ELEC® INSTITUT declines all responsibility if the user does not follow the instructions described in this manual.
- ▶ If for any reason you are in any doubt concerning the use of this device, consult your doctor prior to use.

RESTRICTIONS OF USE

- ▶ The belt must not be connected to another device or any other object other than the BodyBeautiful.
- ▶ The complete device must be used, with all detachable parts in place.
- ▶ Do not touch the inside of the belt when the device is switched on.
- ▶ Do not use the electro-stimulation belt when driving or operating machinery.
- ▶ Do not use in damp conditions.
- ▶ Use the device in a clean environment (dust and dirt-free...).
- ▶ Keep the device away from all sources of heat.
- ▶ Keep the device away from any wave-emitting or high frequency equipment.
- ▶ To reposition the electro-stimulation belt during the session: stop the program in progress, loosen the belt, reposition it and restart once the belt has been readjusted.

WARNINGS

DO NOT USE THIS DEVICE

- ▶ If you carry a pacemaker
- ▶ If you are pregnant
- ▶ During first 6 weeks after baby birth
- ▶ In case of neuromuscular diseases
- ▶ In case of hemorrhagic diseases.

ASK FOR ADVICE FROM YOUR PHYSICIAN :

- ▶ if you have electronic or metallic implants (pins, etc.)
- ▶ in case of skin condition (wounds,...)

MODIFYING THE UNIT IS FORBIDDEN:

It is forbidden to modify the unit without the manufacturer's authorization.

If the unit is altered, a function check and an appropriate test must be carried out to ensure that it can still be used safely.

DO NOT APPLY ELECTRODES:

- ▶ On the heart area, chest and upper back (or passing through the heart).
- ▶ On the face, eyes, mouth, or on the front of the neck.
- ▶ On one side to the other of (or through) the head.
- ▶ potential DANGERS :
- ▶ The simultaneous connection of a patient to a high-frequency surgical unit may cause burns at the points of contact of the stimulator's electrodes, and the stimulator may be damaged.
- ▶ Operation in close proximity of a short wavetherapy unit may cause instability in the stimulator's output power.
- ▶ Applying electrodes close to the thorax may increase the risk of cardiac fibrillation.

V. TECHNICAL SPECIFICATIONS

UNIT :

Class II a medical device in accordance with Directive 93/42/CEE

Type of current	Dual phase
Number of programmes	8 programmes
Number of program variations (or phases)	55
Number of adjustable independent outputs	2
Current	From 0 to 70 mA - From 0 to 1000 Ω
Frequency range / Pulse width range	1 to 100 Hz / 160 to 320 µs
Power	3 x 1,5 V alkaline batteries type LR03 (not provided)
Unit dimensions and weight	84 x 126 x 30 mm - 200 gr
Automatic stop at end of each program	Yes - about 4 min 30 sec
Activation safety	100%
Absence of electrodes or of contact detection	yes detection independent on each of the 2 outputs
Power indicator	bar luminous graph
Weak and/or defective batteries indicator	yes - luminous indicator
Absence contact electrodes Indicator	yes - luminous indicator
Indicator of the programs	luminous pictograms
Temperature of storage / Relative humidity	0°C to + 45°C / Between 10% and 90%
Operating temperature / Relative humidity	+ 5°C to + 45°C / 20% to 65%
Connection device / electrodes	by wires (about 90 cm) + modules with snap fasteners 35mm ← →
Caution	⚠ Read the manual thoroughly
Guarantee	2 years (unit only)

Device in contact with the patient, excluding the heart muscles.

IP22 Protected against the ingress of foreign object ≥ 12.5 mm & against dripping water when tilted up to 15°.

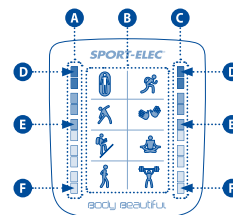
RoHS These units are free of materials likely to have a negative impact on the environment.

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BELT

References.....CT-BB2M-EA
Size (unisex).....from 34 to 60
Length of the unfolded belt.....122 cm
Fabric.....polyester

ADHESIVE ELECTRODES:

Class I medical device (Directive 93/42/CEE)
Adhesive, hydrophilic and hypoallergenic electrodes
References.....EASF110x71
Contact area.....67.5 x 47 mm (x2)
Estimated life expectancy.....40 uses
Please order new compatible adhesive electrodes on www.sport-elec.com.

PROGRAMS DETAILS

Program	1 st Phase			2 nd Phase			3 rd Phase			4 th Phase			5 th Phase			6 th Phase			7 th Phase			Run Time (min)
	F(1)	Lg(1)	T(1)	F(2)	Lg(2)	T(2)	F(3)	Lg(3)	T(3)	F(4)	Lg(4)	T(4)	F(5)	Lg(5)	T(5)	F(6)	Lg(6)	T(6)	F(7)	Lg(7)	T(7)	
1. Preparation *	1	200	4	3	200	4	5	200	4	7	200	4	9	200	4	11	200	4	13	200	4	22
2. Development*	12	300		1	280	8	18	300	20	2	280	10	24	300	12	3	280	12				40
3. Fitness**	20	320	12	40	320	12	3	280	16	70	300	8	100	300	6	8	280	16				20
4. Sudoloris*	4	200	10	5	200	10	6	200	10													20
5. Silhouette Sculpt**	30	320	12	10	280	6	45	320	10	10	280	6	60	320	8	3	280	12				40
6. Recovery*	60	220	0,5	70	220	0,5	80	220	0,5	90	220	0,5	100	220	0,5							20
7. Massage ***	160µs - Frequency variation from 40Hz to 100Hz by increments of 0.5s/Hz => Wobble Time: 30 seconds																					20
8. Muscle Tonic**	70	280	6	3	260	6	85	280	6	3	260	6	100	280	6	3	260	6				20

Caption :

- * Ramp of 1,5s, then all frequencies, then down-ramp 1s for the duration of the program
- ** Up-ramp of 1,6s / Down-ramp of 1,6s before and after each frequency. After the last frequency: recovery of the cycle
- *** Wobulation: each frequency lasts for x seconds. F(2) = F(1) + x Hz, then F(x) = F(x-1) + x Hz
- F(x) Frequency of x phase (Hz)
- Lg(x) Pulse-width of x phase (µs)
- T(x) Duration of x phase (s)

VI. GARANTIE

- ▶ We provide a 24 month guarantee for products supplied by us, from the date of purchase.
- ▶ The belt and adhesive electrodes are for personal use. For hygiene reasons, they can therefore neither be returned nor exchanged. The life expectancy of the adhesive electrodes is estimated at approximately 40 uses.
- ▶ Never attempt to repair or modify the device yourself, or by a repairman not authorized by SPORT-ELEC*.
- ▶ The guarantee becomes null and void if repairs have been carried out by non-authorized persons or if the replacement parts used are not from the same manufacturer.
- ▶ Similarly, only use compatible accessories manufactured by SPORT-ELEC*.
- ▶ During the guarantee period we repair, free of charge, manufacturing defects or faulty parts, reserving the right to decide if certain parts need repairing or replacing, or if the device itself must be replaced.
- ▶ Any damage resulting from improper use (knocks, plugging into a direct current, wrong voltage, ...) does not fall under the guarantee; normal wear and tear does not affect the proper functioning or value of the device.
- ▶ The guarantee is only valid if the date of purchase and the stamp and signature of the retailer appear on the guarantee certificate, or upon presentation of the till receipt. (See «guarantee certificate» at the end of this manual)
- ▶ No other claims fall under the guarantee, unless otherwise provided by law.

ACCESSORIES GUARANTEE :

The belt and electrodes are designed for personal use. For reasons of hygiene, there will be no trade-in nor exchange.

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