FREE ACTION

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HEALTH PROGRAMS

MASSAGE

This program uses frequencies modulated between 3 and 10 Hz for optimum massaging effect.

MUSCLE RELAXING

The frequency range between 0.25 and 3.5 Hz of this program accelerates muscle relaxing.

ACTIVE RECOVERY

Use this program immediately after performing intense efforts. All frequencies generated by «Active recovery» maintain muscle work after effort, enhancing blood flow and the release of endorphins. Each of these frequency sequences is alternated with a relaxing effect.

TENS

This program operates by across-skin nerve stimulation, delivering a range of pulse frequencies in the local area where the electrodes are placed.

ENDORPHINIC

This program relieves muscle tensions effectively.

VASCULARIZATION

This program relieves numb legs sensation.

SPORT PROGRAMS

The Free Action proposes a warming-up phase (X 4 min) before starting the active phase (X) & the recovery phase (A) of the program.

SKIN PREPARATION

Lowers the impedance of the skin barrier. Prepares your skin for the currents emitted by the programs.

RESTORE MUSCLE VOLUME

This program prevents and fights loss of muscle volume resulting usually from a lack of use of a limb following a period of immobilisation.

REINFORCE MUSCLE VOLUME

Apply this program only on muscles with a normal volume. «MUSCLE RESISTANCE» and «MUSCLE ENDURANCE» programs are used to develop muscle strength.

MUSCLE RESISTANCE

The stimulation frequencies aim at increasing the ability to sustain major muscle efforts.

MUSCLE ENDURANCE

After using «MUSCLE RESISTANCE» to acquire the ability to sustain major efforts, use program «MUSCLE ENDURANCE» to extend high intensity muscle efforts over time by maximum stimulation of slow fibres.

MUSCLE FITNESS

Use this program to maintain the level of muscle endurance, resistance, and volume acquired.


EXPLOSIVE FORCE

This program is a logical extension of «REINFORCE MUSCLE VOLUME» and «MUSCLE RESISTANCE». It allows reaching maximum muscle force by stimulating very high speed fibres, with maximum contraction intensity.)

RECOVERY

Using a frequency range between 0.5 and 8 Hz, this program ensures muscles recover faster through relaxing, endorphin-based effects, and increases blood flow.

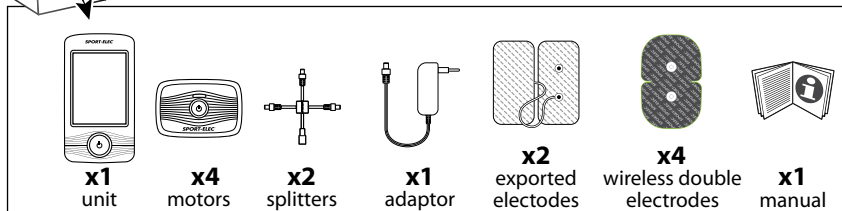I. IMPORTANT INFORMATIONS

You have just purchased an advanced and powerful professional device. You should take the time to familiarise yourself with how this device operates by reading these instructions carefully.

IMPORTANT > Recharge the device completely prior to initial use.

EMERGENCY STOP: Press the "ON/OFF" button (Ⓞ) located on each motor.

THERAPEUTIC APPLICATIONS

- ▶ To strengthen and/or to maintain muscle strength.
- ▶ To ease muscular tensions.
- ▶ To prevent muscular atrophy and to treat pain.

CONTRA-INDICATIONS : Do not use this device :

- ▶ on the heart area
- ▶ If you carry a pacemaker 
- ▶ If you are pregnant 
- ▶ during first 6 weeks after baby birth
- ▶ In case of neuromuscular diseases
- ▶ In case of hemorrhagic diseases.

Ask for advice from your physician :

- ▶ if you have electronic or metallic implants (pins, etc.)
- ▶ in case of skin condition (wounds,...)

PRECAUTIONS OF USE :

- ▶ Do not use the appliance on your face or your neck.
- ▶ Keep away from children.
- ▶ Apply a complete charging cycle if the device is not used for an extended period.
- ▶ Do not repeat two sessions on the same muscle area (max. 40 min., not applicable for HEALTH programs).
- ▶ If you have several FREE ACTION devices, make sure you use the motors connected to the active remote control.

- ▶ Do not use by driving or by working on a machine.
- ▶ Do not use this electro stimulation unit in a damp environment, a bathroom, a sauna, or close to electric machinery connected to mains and earthed, or even piping earthed.
- ▶ The simultaneous connection of a patient to a high frequency surgical unit may cause burns at the points of contact of the stimulator's electrodes, and the stimulator may be damaged.
- ▶ Operation in close proximity of a short wave therapy unit may cause instability in the stimulator's output power.
- ▶ Keep the appliance away from children.
- ▶ Free Action is an electro-medical device. It requires special caution regarding EMC and must be installed and set up according to EMC information provided in this document. Portable and mobile RF devices communications can affect medical electrical equipment. This device should not be used adjacent to or stacked with other equipment with them. In case of it is not possible to do otherwise, it is convenient to check the device while it is running to make sure that it works normally in chosen configuration. The operation of FREE ACTION may undergo interferences due to other devices, even if those fulfill the requirements of emission of the CISPR.

WARNING :

- ▶ It is forbidden to replace the battery.
- ▶ It is forbidden to modify the unit.

II. DESCRIPTION

II.1. MOTOR

See page 7 motor picture

- 1 State indicator
- 2 ON/OFF button - Emergency stop
- 3 Electrode contactors
- 4 Pairing button
- 5 Charging connector

II. 2. UNIT

See page 7 unit picture

- 1 Touch-screen
- 2 ON/OFF button - Emergency stop
- 3 Charging connector

II. 3. ELECTRODES

See page 7 electrodes picture

- 1 Fastener buttons

III. CHARGING

See page 8 how to proceed to charging batteries

IV. START-UP

See page 9 how to proceed to start-up step 1: A = unit ; B = motors

-  Battery charged
-  Low battery
-  Connection status

-  "CONFIRMATION" touchscreen icon
-  "MAIN MENU" touchscreen icon

-  "BACK" touchscreen icon
-  "CHECK MOTOR STATE" touchscreen icon

V. PAIRING THE MOTORS

See page 10 how to pair the motors with the unit
step 2: (Button at the above each motor)
step 3: (Button located underneath each motor)

CONFIGURATION MENU

See page 11 how to access to "configuration" menu
step 2: Adjusting screen luminosity
step 3: Adjusting volume setting
Step 4: Checking the motors status

VI. ADJUSTING POWER

See page 11 how to adjust power of stimulations.

- A** = Reminder of the selected working area
- B** = "zero power" touchscreen icon (press and hold)

IMPORTANT! Before proceeding to power adjustment:

- Attach an electrode to each motor
- Place each electrode (fitted with its motor) on your body

VII. START A PROGRAM

VII.A. INTRODUCTORY PROGRAM

See page 12 how to start the introductory program

Soon as you adjust the power level, the program starts

VII.B. HEALTH PROGRAM

See page 13 how to start a health program

step **5**: Only whether the unit is connected to 3 or 4 motors


VII.C. SPORT PROGRAM

See page 14 how to start a sport program

Step **1**: Access to "sport programs" menu

Step **2**: Selecting the first muscle zone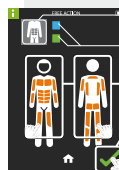

1 After selecting the first muscle area, the tablet displays this screen:

- 1** Indicates the first muscle area (already selected).
- 2** Indicates motor(s) associated to the first muscle area.
- 3** Icons "front" and "back": press again to select a second muscle area.
- 4** Icon "validation": to launch the program directly with all motors on the same area. (go to step **5**).

Step **3**: Selecting the second muscle zone.

Step **4**: Fitting electrodes and validation of muscle zone(s)

Step **5**: Details of program and validation

⌘ = Warm-up phase - ⚡ = Active phase - ⏪ = Recovery phase

Step **6**: Adjusting power of stimulations

Step **7**: Program in progress

VIII. SESSION IN PROGRESS

See "Session in progress" diagram - page 15

- 1.** Motor status check
- 2.** Type of program selected (sport / health)
- 3.** Program selected
- 4.** Working areas (only with sport program)
- 5.** Remaining time for the current phase
- 6.** Phase in progress (warm-up, active phase, recovery)
- 7.** Output current frequency
- 8.** Power indicator for active motors

"PAUSE" FUNCTION

Press the "pause" icon does not interrupt the program, but cancels the frequency. The countdown continues.

SCREENSAVER / SCREEN LOCK

The screen automatically switches to standby during a program. The programme continues. Press the "padlock" icon to unlock the screen.

IX. END OF SESSION

See "End of session" diagram - IX.A.chapter - page 16

"Repeat" touchscreen icon: health programs only

"Add This Program To Bookmarks" touchscreen icon (see Xth chapter).

INTERRUPT A SESSION IN PROGRESS

See "Interrupt a session in progress" diagram - IX.B.chapter - page 16

X. BOOKMARKS / ADD A BOOKMARK

See page 17 (X.A. and X.B. chapter)

XI. TECHNICAL SPECIFICATIONS

SERVICING & CLEANING RECOMMENDATIONS

► Terminals:

The terminals can be cleaned with a cloth soaked in disinfectant.

► Adhesive electrodes:

Rehydrate before and after use.

If necessary, use disinfectant.

Leave on the original support base, in a plastic bag at between 5°C and 10°C.

CONDITIONS FOR USE

- Temperature: 5°C - 40°C (max. 35°C while charging)
- Relative humidity: 20% - 65%


STORAGE CONDITIONS

- Storage temperature: 0°C - 45°C
- Relative humidity in the storage area: 10% - 90%
- Keep in a dry location, away from direct sunlight.

RECOMMENDATIONS:

 Make sure you comply with national regulations when you dispose of your product at end-of-life. Please dispose of this product at a location designed for this purpose in order to ensure the protection of the environment.

CLASS II MAINS POWER

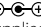▶ Reference: MN0151-E050300
▶ AC 100-240V - 50/60Hz - 0,5A  DC 5V \approx 3A
Only use the mains power unit supplied with the device. Please replace this unit should it fail (contact SPORT-ELEC®, or order at www.sport-elec.com).

ELECTRODE DATA:

Class I medical device (directive 93/42/EEC) - Adhesive, hydrophilic and hypoallergenic electrodes
Reference EASF110X71 - EAEXP100X50
Contact surface 67,5x47mm (x2) - 100x50mm (x2)
Estimated life cycle 40 usages

Do not hesitate to order FREE ACTION adhesive electrodes at www.sport-elec.com.

TERMINAL DATA:

Class IIa medical device (directive 93/42/EEC)
Current type two-phase
Number of programs 14 (8 "SPORT" + 6 "HEALTH")
Number of program variations (or phases) 255
Number of adjustable independent channels: up to 4
 Maximum intensity per channel up to 1kΩ 90mA
Frequency range 0.5-120 Hz
Pulse range 70-360 μs
Power Rechargeable LiPo battery
3.7V - 1020 mAh (remote control)
3.7V - 420 mAh (motors)
Remote control dimensions 79 x 135 x 24 mm
Remote control weight 146 g
Motor dimensions 78 x 52 x 20 mm
Motor weight 51 g
Contact sensor Yes
(channel power gauge > white)
Automatic stop at the end of each session Yes
Start safety Yes: Autotest
 Classification BF type: device in contact with the patient, excluding the heart muscles.
RoHS Devices are constructed without materials with a potential harmful impact on the environment.

XII. SOLVING PROBLEMS

If the problem persists after carrying out the following checks, contact our After-sales service at +33 2 32 96 50 51 or sav@sport-elec.com

MOTOR LED FLASHES RED

▶ Charge (see chapter III) and retry.

TRIANGLE SYMBOL or MOTOR LED FLASHES WHITE

▶ Turn all terminals (remote control + motors) OFF and ON again
▶ If the problem persists, contact the after-sales service directly

WHITE BARGRAPH


The electrode in question is disconnected.

▶ Try new electrodes.
▶ Check the contact between the electrode and the skin.
▶ Check that the motor is fixed to the electrode.
▶ Check that the motor is ON.
▶ Then try to increase power again.

ONE OR SEVERAL TERMINALS FAIL TO COME ON OR EXPECTEDLY TURN OFF

▶ Charge all of the terminals for ~2 hours.
▶ Restart your terminals.
▶ Restart the preferred program.

I CAN'T FEEL ANY MUSCLE CONTRACTIONS

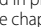▶ Try new electrodes.
▶ Check that the electrodes are in contact with the skin.
▶ Check that the motors are fixed to the electrodes.
▶ Check the connection between the motors and the remote control ("info"  or "motor state check"  touchscreen icons)
▶ Check that a program is indeed in progress (e.g. the "introductory" program  see chapter VII).
▶ Try to increase the power (see chapter VIII)
▶ If you have several FREE ACTION devices, check that you are using the remote control for the motors in question.

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WARRANTY : We accord a 24-month warranty from the date of purchase for the products we distribute. We will repair all manufacturing or material defects free of charge throughout the entire warranty period, however, we reserve the right to decide if some parts must be repaired or replaced, or if the device itself must be replaced. The warranty will not apply in the event of damage caused by inappropriate use (impact, use of direct current, incorrect voltage, etc.); normal usage will not cause prejudice to the satisfactory operation and value of the device. The warranty will expire if repairs are carried out by unauthorised individuals or if spare parts by another brand are used. The warranty will only be valid if the date of purchase and the stamp and signature of the store appear on the warranty card (or upon presentation of the receipt). No other forms of justification will be accepted for the warranty, unless legal provisions specify otherwise.

ELECTRODE WARRANTY : The adhesive electrodes are intended for strictly personal use. Electrodes may not be returned or exchanged for hygiene reasons.

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GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC EMISSIONS

The Free Action is intended for use in the electromagnetic environment specified below. The customer or the user of the Free Action should assure that it is used in such an environment.

EMISSIONS	COMPLIANCE	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
RF emissions CISPR 11	Group 1	The Free Action uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Free Action is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Conform Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Conform	

GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The Free Action is intended for use in the electromagnetic environment specified below. The customer or the user of the Free Action should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines and dc lines	±2 kV for power supply lines	Mains power quality should be that of a typical commercial or hospital environment.
	±1 kV for input/output lines	Not applicable for input/output lines	
Surge IEC 61000-4-5	±1 kV line(s) and neutral ± 2 kV line(s) to earth	±1 kV line(s) and neutral N/A(57.2 EN61000-4-5)	
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % Ur (>95 % dip in Ur) for 0,5 cycle	<5 % Ur (>95 % dip in Ur) for 0,5 cycle	Mains power quality should be that of a typical commercial or hospital environment. If a dips or an interruption of mains power occurs, the current of the Free Action may be dropped off from normal level, it may be necessary to use uninterruptible power supply or a battery.
	40 % Ur (60 % dip in Ur) for 5 cycles	40 % Ur (60 % dip in Ur) for 5 cycles	
	70 % Ur (30 % dip in Ur) for 25 cycles	70 % Ur (30 % dip in Ur) for 25 cycles	
	<5 % Ur (>95 % dip in Ur) for 5s	<5 % Ur (>95 % dip in Ur) for 5s	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE - Ur is the a.c. mains voltage prior to application of the test level

GUIDANCE AND MANUFACTURER'S DECLARATION ELECTROMAGNETIC IMMUNITY

The Free Action is intended for use in the electromagnetic environment specified below. The customer or the user of the Free Action should assure that it is used in such an environment.

IMMUNITY TEST	IEC 60601 TEST LEVEL	COMPLIANCE LEVEL	ELECTROMAGNETIC ENVIRONMENT - GUIDANCE
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3V	Portable and mobile RF communications equipment should be used no closer to any part of the Free Action, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = 1.17 \sqrt{P}$ $d = 1.17 \sqrt{P}$ 80 MHz to 800 MHz $d = 2.33 \sqrt{P}$ 80 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation Distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey ^a should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Free Action is used exceeds the applicable RF compliance level above, the Free Action should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Free Action.

b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

RECOMMENDED SEPARATION DISTANCES BETWEEN PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT AND THE FREE ACTION

The Free Action is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Free Action can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Free Action as recommended below, according to the maximum output power of the communications equipment.

RATED MAXIMUM OUTPUT POWER OF TRANSMITTER (W)	SEPARATION DISTANCE ACCORDING TO FREQUENCY OF TRANSMITTER (M)		
	150 KHZ TO 80 MHZ $d = 1.17 \sqrt{P}$	80 MHZ TO 800 MHZ $d = 1.17 \sqrt{P}$	800 MHZ TO 2,5 GHZ $d = 2.33 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.67	11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 - At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 - These guidelines may not apply in all situations.

Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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